



**HOUND LABS®**

## HOUND® CANNABIS BREATHALYZER SYSTEM RELEASE NOTES

Release Date	September 22, 2023
Release Version	0.1.270
System Downtime	Base station will announce an update is available and recommend installing immediately. Installation is user-initiated following an on-screen prompt to install. Installation should take approximately 5-10 minutes to complete once initiated.
Customer Impact Risk	Minimal
Communicated Externally?	Not Proactively
Downtime Impact	During the software installation, users CANNOT: <ul style="list-style-type: none"><li>• Perform breath collections</li><li>• Connect to Retriever™ for result transmission or receive onboarding information such as users IDs and passwords</li></ul>
Release Summary	<ul style="list-style-type: none"><li>• Connectivity enhancements for stability and usability</li><li>• Capture additional donor demographics to enable Chain of Custody Form (CCF) delivery in the future</li></ul>

### CHANGE: ENHANCEMENTS TO CONNECTIVITY

- **Impact:** Base station improvements made to stabilize connectivity and provide visual indicators to the operator.
  - Connectivity icons in the header will now appear green when connected to Retriever™.
  - Local admins can now navigate to the 'About' screen to determine connectivity status and type – Wi-Fi, cellular, etc.
  - Operator can now connect to hidden networks by entering the name of the network.
- **Purpose:** Ensures stable connectivity and increases usability within the user interface.



## About Screen

**System Admin** 06:30 PDT 09/22/2023

### About

Manufacturer **Hound Labs, Inc.** Mfg Date SN **CNS00001**

Lan IP Address  
Lease Obtained  
Lease Expires

✓ Wifi IP Address **192.168.1.159**  
Lease Obtained  
Lease Expires **Sat Sep 23, 2023 6:29:04 am**

Cell IP Address **10.32.172.47**  
ATT ICCID **Not Available**  
ATT IMSI **Not Available**  
VZW ICCID **Not Available**  
VZW IMSI **Not Available**  
Status **Failed**  
SPN **Not Available**  
IMEI **350344982987701**

Current Software Version **0.1.270**  
Current Firmware Version **0.1.270**  
Available Updates **None**

←] [Home

Connection status shown

When green, connectivity icons indicate connection to Retriever™

## CHANGE: CAPTURE ADDITIONAL DONOR DEMOGRAPHICS

- **Impact:** Adds the ability to capture donor telephone number and email address.
- **Purpose:** Enables future enhancement to send the Chain of Custody Form (CCF) to the donor via text or email.

## Donor Data Entry Screen

**System Admin** 12:43 PDT 09/19/2023

Lab Account Number

Donor ID

Donor First Name

Donor Last Name

Donor Date of Birth

Donor Phone Number

Donor Email Address

Reason for Testing

Remarks

Verified By ▼

Send CCF? ▼

All information fields must be complete to proceed with a collection.