



HOUND LABS®

HOUND® CANNABIS BREATHALYZER SYSTEM RELEASE NOTES

Release Date	October 6, 2023
Release Version	0.2.277
System Downtime	Upon being made available, the base station will automatically download the update and announce that it is available for installation. Installation should take approximately 10-20 minutes.
Customer Impact Risk	Moderate: It is recommended that this update be installed immediately after being made available as it includes necessary system updates to improve overall function and user experience.
Communicated Externally?	This software release aligns with updates to product literature and customer training materials to be made available on October 6, 2023
Downtime Impact	During the software installation, the base station will be unable to conduct any breath collections or other activities.
Release Summary	Improved collection workflow guidance Additional donor information capture Enhanced connectivity functionality display

CHANGE: IMPROVED COLLECTION WORKFLOW GUIDANCE

- **Impact:** Base station screens and language have been updated to provide collectors with appropriate guidance for performing a successful breath collection.
- **Purpose:** Ensures base station provides guidance that is consistent with the procedure described in customer training and product literature.
- Refer to the Hound Labs Collector LMS and [Product Insert](#) for updated collection procedures (<https://houndlabs.com/collector-portal>).



HOUND LABS®

CHANGE: ADDED DONOR INFORMATION CAPTURE

- **Impact:** Added ability to capture donor's preferred method of CCF delivery.
 - A drop-down list is now available to select the donor's preferred method to receive a secure link to the donor copy of the CCF.
 - Menu options include: No, By Text, By Email, By Both
- **Purpose:** Enables the base station to gather information necessary for Retriever™ to send the donor a secure link to the donor copy of the CCF.

System Admin 08:08 PDT 10/04/2023

Lab Account Number 1234567890

Donor ID _____ Verified By ▼

Donor First Name _____

Donor Last Name _____

Donor Date of Birth _____

Donor Phone Number _____ Send CCF? ▼

Donor Email Address _____

Reason for Testing Select Test Reason ▼

Remarks _____

All information fields must be complete to proceed with a collection.

VIEW WORKLIST SCAN CARTRIDGE BEGIN COLLECTION

CCF delivery method selection drop-down list

CHANGE: ENHANCED CONNECTIVITY FUNCTIONALITY DISPLAY


- **Impact:** Added visual feedback in the main header bar of the display to indicate connection status of ethernet, Wi-Fi, and cellular connectivity modes.
 - The header bar now includes icons for ethernet, Wi-Fi, and cellular connectivity.
 - Connectivity mode icons are displayed in one of three colors to indicate status:
 - Gray – connectivity mode is not active.
 - Red – connectivity mode is active but not functional.
 - Green – connectivity mode is active and functional.
 - The active method of connectivity will be surrounded with a green border.



HOUND LABS®

- **Purpose:** Enables the collector to quickly view the status of each connectivity mode, as well as which mode is being used at that time.

Icons show ethernet, cellular, and Wi-Fi connectivity status

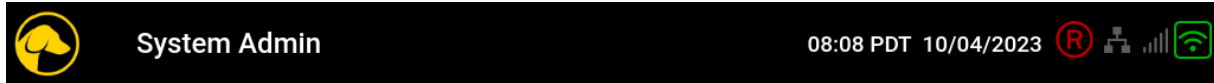

Green border around Wi-Fi icon indicates the active connection



CHANGE: ADDED RETRIEVER™ CONNECTIVITY INDICATOR

- **Impact:** Added visual feedback in the main header bar to indicate connection status with Retriever™
 - The header bar has been updated to include a Retriever™ connection icon.
 - Retriever™ connection icon is displayed in one of three colors to indicate status:
 - Gray – connectivity mode is not active.
 - Red – connectivity is active but not functional.
 - Green – connectivity is active and functional.
- **Purpose:** Enables a collector to quickly view the Retriever™ connection status.



Retriever Connection Icon

