




## HOUND® CANNABIS BREATHALYZER SYSTEM RELEASE NOTES

Release Date	November 16, 2023
Release Version	0.3.286
System Downtime	Base station will announce an update is available and recommend installing immediately. Installation is user-initiated following an on-screen prompt to install. Installation should take approximately 15-20 minutes to complete once initiated.
Customer Impact Risk	Minimal
Communicated Externally?	Not proactively
Downtime Impact	During the software installation, users CANNOT: <ul style="list-style-type: none"><li>• Perform breath collections</li><li>• Connect to Retriever™ for result transmission or receive onboarding information such as users IDs and passwords</li></ul>
Release Summary	<ul style="list-style-type: none"><li>• Capture collector remarks</li><li>• Improve collection workflow with 'begin collection' + 'collection complete' screens</li><li>• Text messaging disclaimer</li></ul>

### CHANGE: ENHANCEMENT TO CAPTURE COLLECTOR REMARKS

- **Impact:** Base station user interface improved to capture collector remarks at any time during the collection workflow
- **Purpose:** Aligns more closely with paper CCF process where a collector can capture a remark throughout the collection process

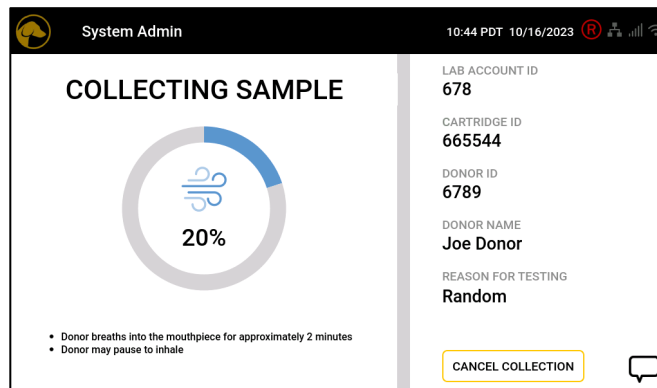


Icon to enter a remark now available on all screens



# HOUND LABS®


## CHANGE: USER INTERFACE UPDATE TO INDICATE 'BEGIN COLLECTION'

- Impact: Adds a screen to the beginning of the collection workflow
- Purpose: Ensures the collector is aware that the donor should begin breathing

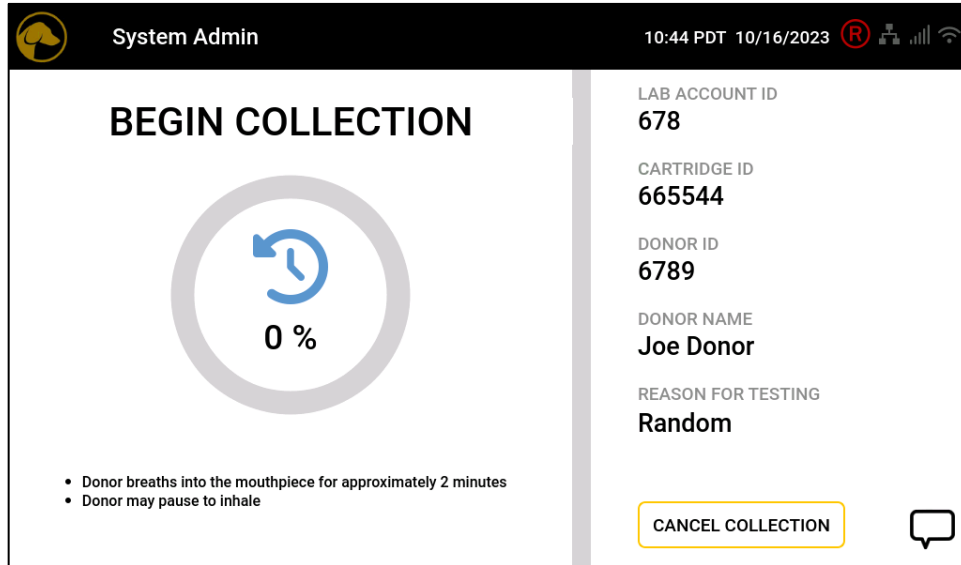

## CHANGE: USER INTERFACE UPDATE TO INDICATE COLLECTION COMPLETE


- Impact: Adds a screen to the end of the collection workflow
- Purpose: Ensures the collector is aware when the collection is complete to progress in the collection workflow

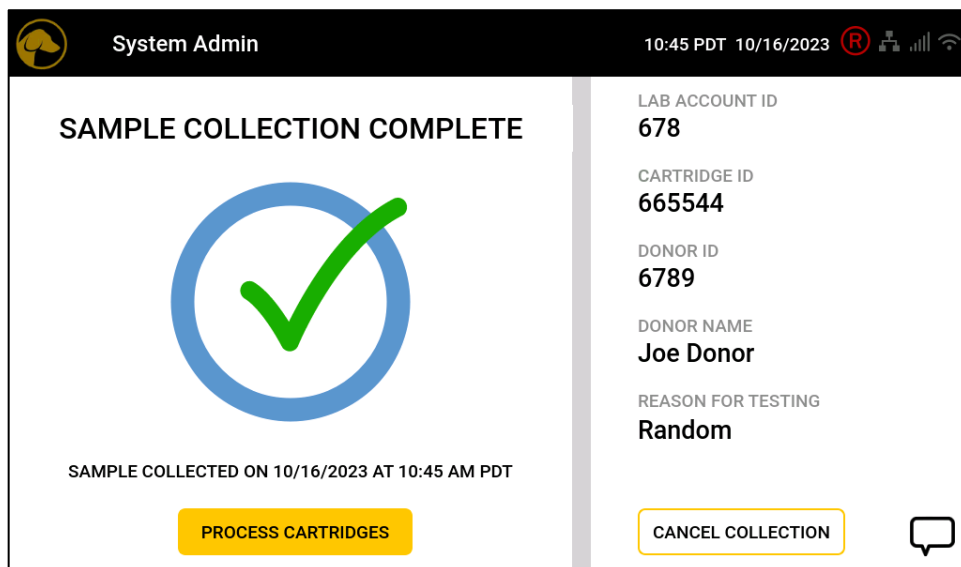




## CHANGE: USER INTERFACE UPDATE TO INCLUDE TEXT MESSAGING DISCLAIMER

- **Impact:** Adds a disclaimer screen to the collection workflow
- **Purpose:** Informs donor about standard text messaging rates when a donor selects to receive their CCF by text

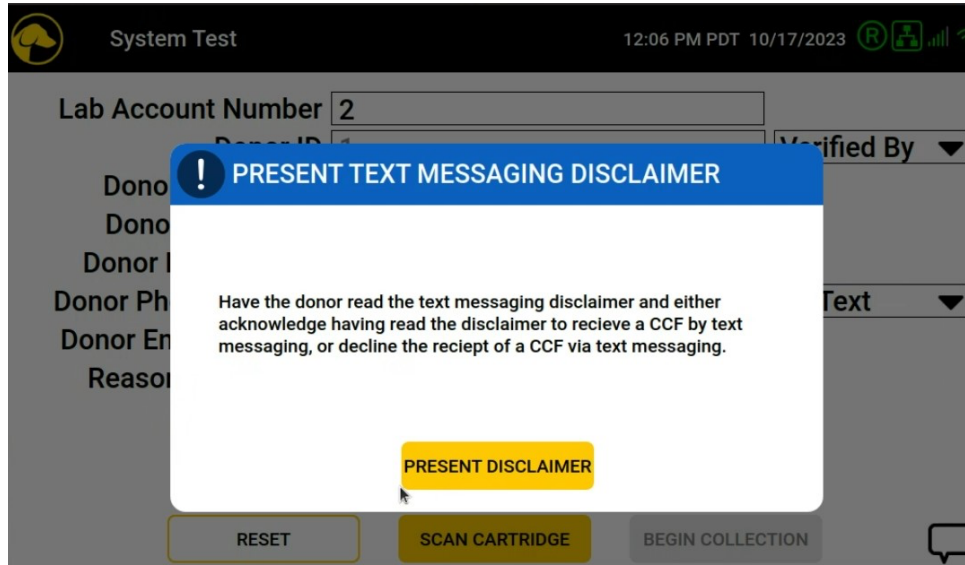

*Message presented to collector prior to donor acknowledging text message disclaimer*

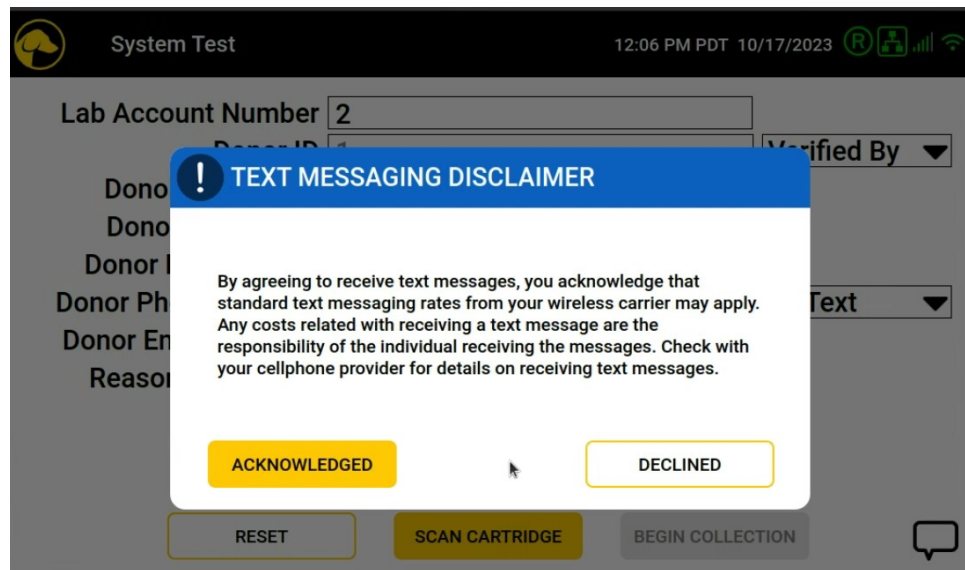

*Text message disclaimer to be acknowledged by donor*